

CE



CHRIS HANLEY & PARTNERS
4 Kilvey Road, Brackmills, Northampton NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk



ACHILLES WEDGE INSTRUCTION LEAFLET



Features

- 4" (10cm) neoprene padded foam wedge
- Fits into most closed heel walkers
- Multi-layered for progressive tendon stretching
- Each section is $\frac{3}{4}$ " (2cm) in height and may be "peeled away" during Achilles Tendon rehabilitation to achieve the desired height.

Indications

- Achilles Tendon rehabilitation
- Post-trauma
- Post-operative



Fitting Instructions

- Peel away sections of the wedge to reach the desired angle if required
- Place wedge into the walker
- Remove another section every two weeks while rehabilitation progresses.
- Patient can then use the single, smallest wedge in their own footwear for one or two weeks
- Always following the guidance from your physician or Orthotist when using this product.

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.