

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS
4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



BOXIA ANKLE BRACE (including Shoeless Wrap) INSTRUCTION LEAFLET




Fitting Instructions

1. Undo the shoelaces of the shoe and place the shoe insert between the tongue of the shoe and the shoelaces (narrowest end towards the toes). Re-do the shoelaces.



2. With the Achilles tendon gel pad at the bottom, wrap the support around the leg, just above the ankle, and fasten snugly with the Velcro strapping.

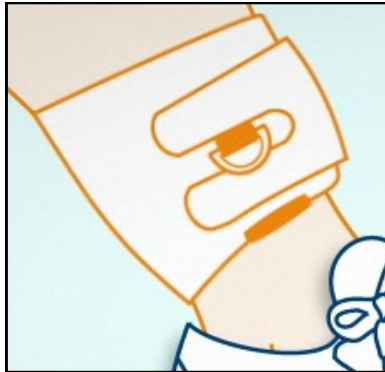

3. Bend the traction elastic, feed through the D-ring and attach to the hook on the shoe insert



4. Pull the elastic to obtain the desired traction and secure on the ankle support with the Velcro fastenings



To use the Boxia Ankle Brace without footwear, you will need to fit the

Shoeless Wrap:

1. Place the wrap around the foot, (ensuring that the hook is pointing towards the toes) and secure with the Velcro straps.



2. Follow the instructions on the previous page from No. 2 onwards

Care and Maintenance

- Close all Velcro fastenings before washing
- Hand wash only in cool water with mild detergent
- Use a dry towel to absorb excess moisture and leave to dry at room temperature away from direct heat. Do not iron.
- Do not allow the product to come into contact with alcohol or solvents

Precautions

- It is advisable to wear a sock underneath the orthosis to prevent chafing
- Inflammable - keep away from fire
- Always follow the guidance of your physician or Orthotist regarding the use of this orthosis