

Precautions

- Carefully read use instructions and warnings prior to use
- If you experience increased pain, swelling, sensation changes or any adverse reactions while using this product, contact your medical professional immediately
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also essential parts of a safe treatment program.

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS

4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057



EZ ROM INSTRUCTION LEAFLET



Fitting Instructions

1. Apply the foam wraps


- Remove foam cuffs from hinge assemblies
- Apply calf and thigh foam cuffs snugly around the leg, closing on the front of the leg

2. Align and apply hinges

- Position and press the lateral side bar assembly onto the cuffs, along the midline of the leg. Centre the hinge pivot axis slightly above the knee joint line
- Repeat for the medial side bar assembly, ensuring that the hinge pivot axis align vertically
- Reposition the foam cuffs as needed, ensuring that the patella remains exposed

3. Secure the leg straps

- Secure and fasten all straps, starting with the straps closest to the knee

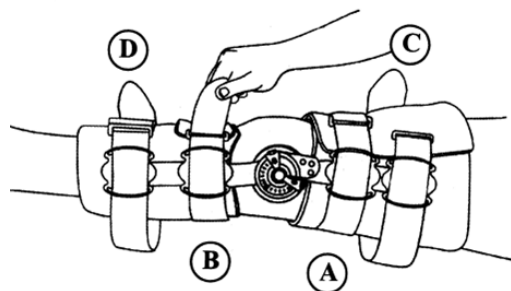

Brace Application and Removal

A properly applied brace will have:


- Foam wraps secured at the front of the leg
- Hinges aligned vertically with each other at the midline of the leg and pivot just above the knee axis
- All straps secured at the front of the leg

Removal and Re-application:

- To remove the brace, unfasten the straps and open the foam pads. The brace can then be removed and reapplied as a single unit.


Hinge Operation and Adjustment

To adjust the hinge, press the flexion or extension button and rotate until the desired degree of flexion or extension shows through the corresponding window and release button. **The “0-lock” settings are used to lock the brace in full extension.**

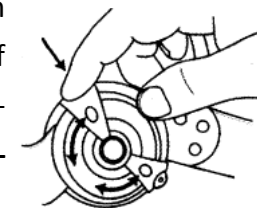

Enforced Compliance

- Loop the enclosed lock ties through the hole in the flexion and extension buttons to limit range of motion

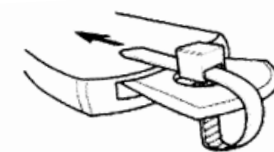

Side Bar Adjustment

1. To breakdown the side bars, place the hinge on a flat surface with the scored line just off the edge and firmly press down to snap off
2. Slide the End Cap D-ring over the bar to cover the broken edge
3. Peel off the backing from the PSA Hook and attach to the back of the bar, making sure the holes are aligned, and press firmly
4. Press the End Cap Rivet firmly through the hook, the bar and the End Cap D-ring until you hear a click.

Use and Care

- All foam pads may be trimmed as necessary
- Hand wash foam liners with mild soap
- Air dry only - do not use heat
- Always consult your clinician or therapist before making any adjustments to the brace