

Ordering Information

Size	Code
Universal	HAS

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS
4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



HIGH ARM SLING INSTRUCTIONS



Indications

Indications for use:

- Post surgery
- Hand surgery
- Post trauma

Fitting Instructions

The High Arm Sling is universal for both left and right arms. These instructions for use demonstrate the product in a right arm application, however the left arm configuration is an identical application process.



1. Place the injured arm into the sling, ensuring the elbow is held securely in place.



2. Whilst holding the sling in a closed position, attach the securing strap to the front of the sling as shown and fold over to secure.



3. To attach the D-ring strap, ensure it is positioned 5-7cm up from the patients wrist. Fold over to secure, ensuring that the side facing D-ring is positioned on the inside of the sling.




4. Place the shoulder strap over the uninjured shoulder and feed through the D-ring attachment, securing the strap firmly with the Velcro® fastening as shown.

Adjust the height as required.

PLEASE NOTE: To use as a shoulder immobiliser, please follow these additional instructions:



1. Attach the immobilisation strap to the front of the High Arm Sling, 3-5cm away from the securing strap.

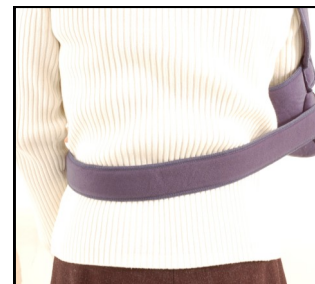

2. Place the immobilisation strap around the back of the waist.



3. Feed the immobilisation strap through the side facing the D-ring (as shown) and secure with the Velcro® fastening