

CE



**CHRIS HANLEY & PARTNERS**  
4 Kilvey Road, Brackmills, Northampton NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)



# HINGED KNEE WRAP INSTRUCTION LEAFLET



## **Features**

- Front opening, wrap around design
- Open patella and popliteal area for added comfort during flexion
- Available in two lengths - Standard (30cm) and Long (40cm)
- Standard length is also available in breathable, black Orthotex material
- Touch and close Velcro fastening
- Enclosed hinged side steels
- Open hinge pockets for easy side steel removal
- Universal fitting

## **Indications**

- Gives increased medial / lateral stability
- Provides compression and therapeutic warmth
- RA / OA
- Mild to moderate hyperextension and knee instability

## **Precautions**

- Do not use if heat causes allergic reactions in the patient

## **Fitting Instructions**

1. Undo the tabbed Velcro straps and open up the wrap
2. Wrap around the leg with the inside label at the top and the popliteal opening at the back, making sure that the hinges line up with the knee cap and that it is visible through the opening
3. Fasten the wrap using the Velcro straps to ensure a snug fit around the leg.

## **Care and Maintenance**

- Remove the side steels before washing
- Wash liner in cold water on a gentle cycle with mild detergent
- Do not dry clean or tumble dry
- Do not use bleach
- Dry away from heat

## **Single Patient Use**

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.