

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS

4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



HINGED SWEDISH KNEE CAGE INSTRUCTION LEAFLET




Introduction

The Hinged Swedish Knee Cage is a 3-point pressure knee brace, based on the original Swedish Knee Cage, which is designed to limit genu recurvatum. The hinge allows the longer uprights to fold back and not protrude when sitting.

Fitting Instructions

1. Select proper size from the chart
2. Fit with patient seated, knee flexed at 90°
3. Orient knee cage with longer uprights proximally
4. Loosen all straps
5. Apply knee cage as proximal as metal calf band will comfortably allow at 90°
6. Align proximal uprights to centre of thigh
7. Tighten distal anterior thigh strap
8. Tighten distal anterior shin strap until distal upright is centred on lower leg
9. Tighten proximal posterior thigh strap until proximal upright is centred on thigh
10. Tighten proximal anterior thigh strap
11. Tighten distal posterior thigh strap

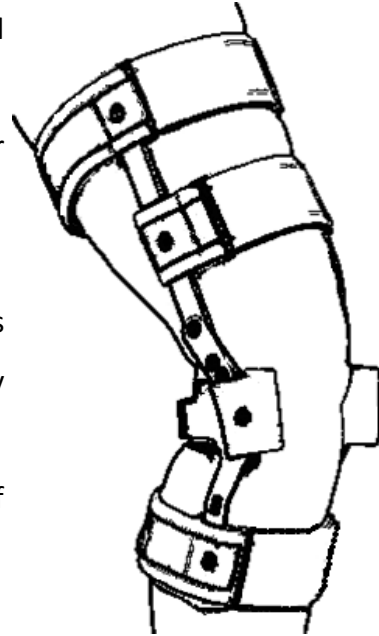

Size Chart

Part Number	Size	Circ. 4" above knee
KCH-S	Small	11" - 15"
KCH-R	Regular	15" - 21"

Adjustments

The instructions above should provide a lock-out at 10—15° of flexion. If less flexion is required proceed with a combination of the following as needed:

1. Loosen posterior knee strap by repositioning the 2 pan head attachment screws anteriorly (1 position anteriorly allows approximately 5° more extension).
2. Loosen anterior distal shin strap to align distal upright posterior to centre of lower leg.
3. Loosen anterior distal thigh strap.