

## Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.


CE



**CHRIS HANLEY & PARTNERS**  
4 Kilvey Road, Brackmills, Northampton NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)



# LERMAN MINERVA CERVICAL ORTHOSIS INSTRUCTION LEAFLET

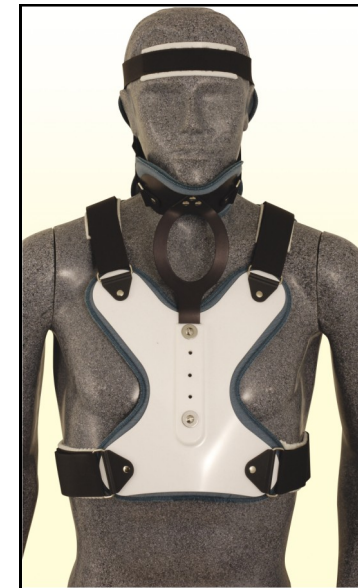

## Features

- Controls flexion, extension, lateral and rotary movements of the cervical spine
- Two shells, anterior and posterior, which are joined by torso and shoulder straps
- Posterior section has movable, shaped occipital support
- Sliding mandibular support is attached to the chest plate with trachea opening
- Forehead strap for additional support and stabilisation
- Silicone chin pad for greater comfort
- Entire brace MRI safe

## Indications

- Stabilisation (C2 - T2)
- Fractures
- Post trauma, muscle and ligament injury
- Nerve impingement
- Post surgery
- RA / Spondylitis

## Application

1. Compare the posterior section with the patient to roughly determine the correct height of the occipital support with respect to the posterior shell. Remove the screws and make slide adjustments as necessary. Replace the screws.
2. With the help of an assistant, log-roll the patient to his/her side to fit the posterior section. Be sure the head and neck are supported during this process. When a successful posterior fit is achieved, roll the patient onto his/her back.
3. Contour and adjust the anterior section as needed.
4. Adjust all hook and pile closures to a snug comfortable position.

