

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS
4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



MALLEOSTRONG ANKLE BRACE INSTRUCTION LEAFLET



Fitting Instructions

1. Open all the Velcro straps and insert the forefoot into the ankle brace (A)



2. Align the shells on the side of the ankle and fasten the lower rear strap (B)



3. Tighten the diagonal strap at the front (C)



4. Pass it around the brace (D)



5. Then fasten with the Velcro strap so that the ankle brace fits snugly (E)



Use and Care

- Handwash in lukewarm water (30°) with mild soap and rinse thoroughly
- Dry away from direct heat

Precautions

- Make sure that the pressure exerted by the device does not impinge on injured or swollen parts of the body
- Flammable - stay away from fire
- If in doubt about the method of application, contact an Orthopaedic expert