

Removal

- Before removing the collar, mark the position of the Velcro strap location to ensure proper fit is maintained after reapplication
- With the collar secured on the patient, unfasten the Velcro strap on the RIGHT side and then the LEFT side. Remove collar from patient.
- Remove liner in a similar manner as it was applied.

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS
4 Kilvey Road, Brackmills, Northampton NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk



PHILADELPHIA[®] COOLMAX[®] LINER INSTRUCTIONS



Features

- Specifically designed for the Philadelphia® Cervical Collar to enhance the patient's level of comfort during periods of extended wear
- Made of 91% coolmax, 9% lycra for optimal comfort and absorption
- Trachea opening
- Sanitary and soft
- Form fitting sizes to ensure easy application
- Washable and reusable
- Sizes available to form fit all Philadelphia® Cervical Collars.

Sizes and Cleaning

Philadelphia® Coolmax® Liner sizes are identical to Philadelphia® Cervical Collar sizes. (Fits all collar heights)


	Small	Medium	Large
Code	PCL-S	PCL-M	PCL-L

The Philadelphia® Coolmax® Liner is washable and reusable. It is important to clean the liner of bodily fluids and contaminants after each use. Simply wash in warm water with mild soap. Line or tumble dry.

Application

Each Philadelphia® Coolmax® Liner is individually designed to fit securely on the Philadelphia® Cervical Collar. After matching the liner size to the corresponding collar, insert the collar into the liner as follows:

1. Align the top end of the collar with the upper edge of the liner seam. Insert the collar into the liner.

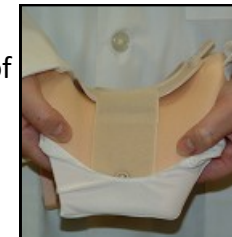

2. Simply follow around the edge of the collar with the liner seam. The seam should finish at the opposite end of the collar.



3. With both hands, place the top portion of the liner over the collar. Complete by tucking the label under the liner. For proper fit, the liner label and central seam must be parallel to the plastic reinforcement.



This application is identical for both the front and back portions of the Philadelphia® Cervical Collar.

4. Once the liner is secured onto the collar, follow the collar application instructions, as provided by a qualified healthcare professional.