

Precautions

- If you experience sudden neck or back pain, numbness or tingling, contact your doctor
- If you have any problems or questions about the use and care of this product contact your orthotic professional.

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS

4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



PHILADELPHIA CERVICAL COLLAR INSTRUCTIONS



Measurement and Sizing

Proper sizing is important for patient immobilisation and comfort. Always maintain the patient's head in neutral alignment.

Height: Measure the vertical distance from the top of the patient's shoulder to the tip of the chin (Fig. A). If the patient's measurement falls between two consecutive sizes, apply the smaller size first.

Circumference: Measure around the patient's neck (Fig. B).



Size Chart & Order Codes

Circumference	Height			
	2¼ (57mm)	3¼ (82mm)	4¼ (108mm)	5¼ (133mm)
<i>Small</i> Cir: 10-13" (25-33cm)	PC2-S	PC3-S	PC4-S	PC5-S
<i>Medium</i> Cir: 13-16" (33-41cm)	PC2-M	PC3-M	PC4-M	PC5-M
<i>Large</i> Cir: 16-19" (41-48cm)	PC2-L	PC3-L	PC4-L	PC5-L

Application

1. After selecting the properly sized collar, apply the back piece from the collar to the back of the patient's neck. Centre the collar. The arrow should point upward.
2. Apply the front piece of the collar with the chin secured in the recess. Centre the collar to secure neutral alignment. The front piece overlaps the back piece to ensure effective immobilisation and comfort. The arrow should point upward.
3. With hook and loop fasteners, tighten the collar with a bilateral adjustment. This will secure the patient's cervical region in neutral alignment.



Features

- Non-toxic Plastazote® Foam
- Total Cervical Arch Support
- Full range of sizes for custom fit
- Hook and loop fasteners
- X-Ray lucent
- Easily cleaned with soap and water