

Care and Maintenance

- All pads are attached by hook and loop and can be removed for washing
- Wash pads by hand with mild detergent
- Allow to air dry
- Clean the frame by hand with mild detergent or alcohol, then rinse with water and dry before reapplication

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS

4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



ROM WHFO INSTRUCTION LEAFLET



Features

- Anatomically molded polyethylene wrist-hand-finger platform and forearm sections, connected by an adjustable wrist hinge
- Limits range of motion of the wrist while giving padded total contact support to the thumbs, fingers and forearm
- Bilateral wrist joints allow for selecting flexion and extension ranges between 0-35°
- May be set for progressive positioning
- Soft foam liner increases patient tolerance
- C-fold hook and loop straps are easily adjustable

Indications

- Post-operative conditions
- Post-cast management
- Arthritis
- Soft tissue contracture management

Fitting Instructions

1. Before attempting to apply the orthosis to the patient, spread the plastic wide at the dorsal openings at both forearm and hand sections
2. Position the splint so that the articulation points of the wrist coincide with the anatomical wrist joint
3. Proximal and distal components may be positioned by loosening the Chicago screws, then sliding the components and re-tightening
4. You can set the range of motion by placing the adjustment screws in the desired position. Make sure that both sections of screws are tightened together using the supplied Allen wrench
5. If loosening of the screws occurs, use Loctite or white glue to secure the settings

Possible Fitting Problems

Pay close attentions to fit on patients who are comatose, have open wounds or pressure areas, or are without full mental capabilities