

## Care and Maintenance

- Wash liner in cold water on a gentle cycle with mild detergent
- Do not dry clean or tumble dry
- Do not use bleach
- Dry away from heat
- Plastic parts may be wiped clean with a damp cloth

## Precautions

Do not use if heat causes allergic reactions in the patient

## Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



**CHRIS HANLEY & PARTNERS**

**4 Kilvey Road, Brackmills, Northampton NN4 7BQ**

**T: 01604 709999 F: 01604 700057**

**www.chaneco.co.uk**



# SHELL AIR / SHELL WALKER INSTRUCTION LEAFLET


**(Available in Standard 17" or Short 11" versions)**



## Fitting Instructions



1. Place the foot and leg inside the foam liner and fasten it snugly.

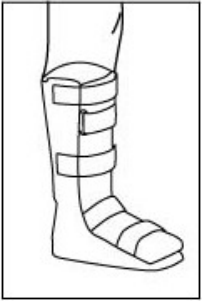


2. Spread the uprights and place the wrapped foot into the walker, making sure the bilateral uprights are in the correct position, in line with the malleoli.



3. You may use the foam pads to fill in any spaces between the liner and the walker for greater comfort.



4. Place the front and back panels in their corresponding places.

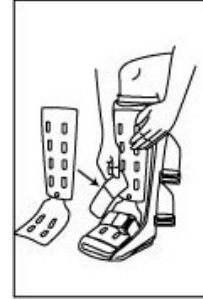

5. Firmly fasten all the straps, starting with the strap closest to the toes.

## Adjusting the Aircell Pressure - SHELL AIR WALKER ONLY

For additional support, inflate the aircells by inserting the pump into the tube.

**NOTE:** The pump should be firmly inserted into the tube to push the air valve down. Inflate with the transparent tip and deflate with the black tip.

