

## Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



**CHRIS HANLEY & PARTNERS**  
4 Kilvey Road, Brackmills, Northampton NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)



# SHOULDER ABDUCTION WEDGE INSTRUCTION LEAFLET



## Features

- High density foam wedge
- Choice of positioning in 45° or 90° abduction
- Swathe design for maximum stability
- Upper and lower arm cuffs for effective immobilisation
- Padded shoulder straps for enhanced comfort
- Simple touch-and-close Velcro fastening system

## Indications

- Post trauma
- Post surgery

## Patient Information

- This product should be used only on the recommendation of a suitably qualified medical professional
- The initial fitting should be carried out by a suitably qualified medical professional
- You should follow the instructions and advice given to you regarding use of this product

## Application

1. Place the wedge underneath the arm to the desired abduction  
- either 45° or 90°
2. Attach one end of the shoulder strap to the front flat panel of the wedge
3. The shoulder strap should come across the patient's opposite shoulder and attach to the flat back panel of the wedge
4. Wrap one of the cuffs around the patient's forearm and the other on the upper arm



5. The cuffs will then attach to the wedge using the Velcro for increased stability
6. The swathe is then wrapped around the wedge, around the abdomen and under the opposite arm