



# TRACKER KNEE BRACE INSTRUCTION LEAFLET



CE



**CHRIS HANLEY & PARTNERS**  
4 Kilvey Road, Brackmills, Northampton NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)



## **Features**

- Ergonomically designed, AirX™ brace with polycentric hinges that provide medial and lateral support
- Reversible yoke for either medial or lateral stabilisation to the knee and patella
- Padded full patella buttress
- Popliteal panel made from elastane to prevent bunching at the back of the knee
- Integral padding on either side of the knee
- Controls mild hyperextension

## **Indications**

- Post trauma
- Post surgery
- Anterior and general knee pain
- Patella dislocation

## **Precautions**

- Make sure that the pressure exerted by the device does not impinge on injured or swollen parts of the body - should fit firmly not tightly
- Flammable - stay away from fire
- Do not allow contact with oil, ointments or solvents
- If in doubt about the method of application, contact an Orthopaedic expert
- Remove immediately if the product causes swelling, heat rash or allergies

## **Fitting Instructions**

1. Remove the patella stabilising strap so that the brace can be pulled on over the knee
2. Locate the patella within the cut-out
3. Ensure that the hinges centres are aligned with the knee joint
4. Apply the patella stabilising strap using the Velcro fastening. A directional pull can be applied to the patella by tightening the strap in the required direction.

## **Use and Care**

- Remove metal hinges before washing
- Handwash the softgoods in lukewarm, soapy water (30°) and rinse thoroughly
- Do not use detergents or bleach
- Dry naturally, away from direct heat

## **Single Patient Use**

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician. Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.