

Care and Maintenance

- Remove the aluminium stays before washing
- Wash liner in cold water on a gentle cycle with mild detergent
- Do not dry clean or tumble dry
- Do not use bleach
- Dry away from heat
- Plastic parts may be wiped clean with a damp cloth

Precautions

Do not use if heat causes allergic reactions in the patient

Single Patient Use

Any product supplied by Chris Hanley & Partners is for use by a single named patient only, as prescribed and fitted by a qualified clinician.

Re-use of any product is potentially dangerous and will invalidate the CE mark and any warranty.

CE



CHRIS HANLEY & PARTNERS

4 Kilvey Road, Brackmills, Northampton NN4 7BQ

T: 01604 709999 F: 01604 700057

www.chaneco.co.uk



TRI PANEL KNEE IMMOBILISER INSTRUCTION LEAFLET



Features

- 3 panels of soft foam laminated material for comfort and exact fit
- Straight lateral and medial aluminium stays provide increased stability
- Velcro closure for secure fit
- Detachable side panels can be angled for a customised fit

Indications

- Post trauma
- Post surgery
- Immobilisation for pain relief
- Cast removal

Size Charts & Order Codes

20" Tri Panel Knee Immobiliser

Size	Knee Joint Circ	Code
Small	13-14" (33-36cm)	KI-20-S
Medium	14-15" (36-38cm)	KI-20-M
Large	15-16" (38-41cm)	KI-20-L
X-Large	16-18" (41-46cm)	KI-20-XL

Universal Tri Panel Knee Immobiliser


Size	Knee Joint Circ	Code
12"	14-16" (36-41cm)	KI-UNI-12
14"	14-16" (36-41cm)	KI-UNI-14
16"	14-16" (36-41cm)	KI-UNI-16
18"	14-16" (36-41cm)	KI-UNI-18
20"	14-16" (36-41cm)	KI-UNI-20
22"	14-16" (36-41cm)	KI-UNI-22
24"	14-16" (36-41cm)	KI-UNI-24

Fitting Instructions



1. Refer to the sizing charts to make sure that you have the correct size.


Undo all the straps and remove the side panels.

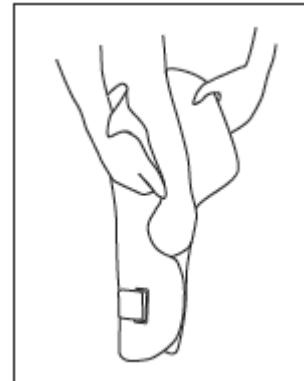

2. Wrap the sleeve around the leg, making sure that the knee cap is visible through the opening.

Fasten the sleeve using the Velcro straps to ensure a snug fit around the leg.

Contour the side panels to shape of the leg by bending the aluminium stays and then reattach to the sleeve.

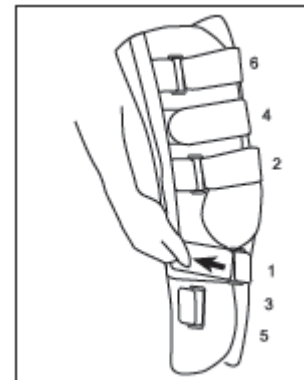

3. Fasten the straps through the D-rings in the order indicated