



The following should be read thoroughly read before use:

Important Notes:

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

Intended User:

For use by both suitably trained healthcare professionals in a healthcare establishment and ‘Lay Users’ in a home use environment.

Intended Purpose:

Provides firm support to back and abdominal area during pregnancy.

Indications For Use:

- Pubic Symphysis pain
- Sacro-iliac pain
- Helps relieve lower back pain during pregnancy
- Can be used during post-natal recovery

Product Characteristics / Features:

- Easily adjustable contoured support belt with hook and loop closure
- This maternity belt is made from durable elastic strapping material for the patient comfort and support
- The depth of the belt at the rear is 15cm which tapers to 13.5cm at the front to fit snugly under the patients tummy to help to support it
- Low profile shape for improved fit when sitting

How it’s supplied:

It is supplied non-sterile and ready for single patient use.

Devices Required For Use, And Interoperability

Requirements:

None.

Storage Conditions:

Store in a cool and dry environment.

Care and Maintenance:

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not expose to alcohol, ointments or solvents
- The device should be safety checked by the providing service and/or user every 6 months.

Contraindications:

None.

Precautions / Warnings / Residual Risks:

Contains latex.

If allergic skin rashes occur, please consult your healthcare professional.

Possible Adverse Reactions:

Contains latex.

If allergic skin rashes occur, please consult your healthcare professional.

Single Patient Use Precaution:

This device is designed and sold for single patient use only.

The effects of any unauthorised reuse can result in the following complications:

- Risk of Cross contamination;
- Risk of mechanical fatigue, and/or associated failure.

Directions For Use / Fitting Instructions:

- Place the maternity belt over the sacro-iliac joints at the back and wrap the maternity belt around the pelvis, directing either side of the maternity belt around the lower abdomen and across the pubic bone
- Close the belt at the front using the hook and loop fastening
- Finally make sure the maternity belt has enough support for the patient and that it is not too tight

Disposal

Discard after single patient use and dispose of as common waste according to national guidelines.
















Serious Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Sizes & REF / Codes

Size	Abdomen Circumference		Code
	Inches	CM	
X Small	35 - 38	90 - 96	MBD-XS
Small	38 - 40	90 - 102	MBD-S
Medium	40 - 42	102 - 108	MBD-M
Large	42 - 46	108 - 116	MBD-L
X Large	46 - 49	116 - 126	MBD-XL
XX Large	49 - 53	126 - 136	MBD-XXL
XXX Large	53 - 57	136 - 146	MBD-XXXL
XXXX Large	57 - 61	146 - 156	MBD-XXXXL

LABELLING SYMBOLOGY
EN ISO 15223-1:2021, unless otherwise stated

	Manufacturer		Consult Instructions for Use		Unique Device Identifier
	Authorised representative in the European Community		Caution		Single Patient Use
	Medical Device		Non-Sterile		Quantity - Abbreviation
	Catalogue Number		CE Mark - EU Regulation 2017/745		Contains Latex
	Batch Code		UKCA Mark - UK Regulation SI 2002 No 618		Date of Manufacture

MBD IFU—Revision No: 6 - September 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057



Ukalia Ltd
Clockwise River, 48-60 King St,
Belfast, BT1 2BE
Northern Ireland
info@ukalia.com

