


Care and Maintenance

- Wash at 40° with mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Double Shoulder Brace Fitting Guidelines & Care Instructions

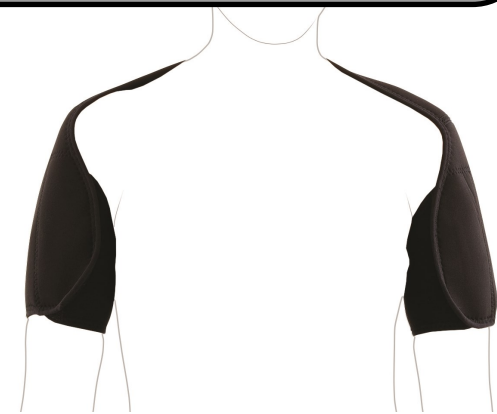

DSS IFU - Revision 5 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire
Retardant



Features

- Maintains the head of the humerus in the Glenoid Fossa
- Durable, comfortable neoprene construction
- Provides therapeutic warmth to the shoulder joint
- Adjustable two-part touch and close fastening design
- Washable at 40°

Indications

- CVA
- Post-trauma - Muscle/ligament strain
- Post-surgery - Shoulder instability
- Joint capsule laxity

Size Table

Size	Chest Circumference		Code
	Inches	CM	
Small	26 - 32	68 - 81	DSS-S
Medium	32 - 38	81 - 96	DSS-M
Large	38 - 44	97 - 112	DSS-L
X Large	44 - 50	112 - 127	DSS-XL

Fitting Instructions

- Select the right size based on your chest circumference
- Slide the sleeve up your arm towards the shoulder region
- Wrap the brace across your shoulders and attach around the other arm, securing with the hook & loop fastening
- Adjust the initial arm as necessary for comfort and support

Additional Information

- Material content: SBR 70%, Nylon 25%, Elastic 5%
- Contains Latex