



The following should be read thoroughly before use:

**Important Notes:**

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

**Intended User:**

For use by both suitably trained healthcare professionals in a healthcare establishment and 'Lay Users' in a home use environment.

**Intended Purpose:**

Provides support for the arm.

**Indications For Use:**

- Post-Surgery
- Post-Trauma
- CVA

**Product Characteristics / Features:**

Key functional elements include:

- Cost effective support for the arm
- Thumb loop limits wrist drop and holds the hand in position
- Ventilated nylon mesh provides excellent air circulation
- Height adjustable webbing strap with touch and close fastening
- Universal - fits right or left arm

**How it's supplied:**

It is supplied non-sterile and ready for single patient use.

**Devices Required For Use, And Interoperability Requirements:**

None

**Storage Conditions:**

Store in a cool and dry environment.

**Care and Maintenance:**

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature.
- Do not expose to alcohol, ointments or solvents.
- The device should be safety checked by the providing service and/or user every 6 months.

**Contraindications:**

None

**Precautions / Warnings / Residual Risks:**

If allergic skin rashes occur, please consult your healthcare professional.

**Possible Adverse Reactions:**

None

**Directions For Use / Fitting Instructions:**

The Mesh Arm Sling is universal for both left and right arms. These instructions will demonstrate the product for a right arm application, however the left arm configuration is an identical application process.

- Remove the sling from its plastic packaging and unfold it.
- With the curved section of the sling facing your **left** arm, place the strap of the sling over your head and pass your **right** arm through it.
- Put your **right** forearm into the sling and position the strap so the padded area is over your left shoulder. (The pad should be facing upwards with the strap visible on top)

**Adjusting your sling:**

Your arm should be positioned in the right angle position as pictured. If it is not you can adjust your Mesh Arm Sling by moving the hook & loop adjustment strap as follows.



Adjustment Strap

- Undo the strap with your left hand.
- Pull or release the strap until your right arm is in the right angle position.
- Firmly replace strap.



### Single Patient Use Precaution:

This device is designed and sold for single patient use only. The effects of any unauthorised reuse can result in the following complications:  
 Risk of Cross contamination;  
 Risk of mechanical fatigue, and/or associated failure.

### Disposal

Discard after single patient use and dispose of as common waste according to national guidelines.
















### Serious Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### Sizes & REF / Codes

Size	Forearm Length		Code
	Inches	CM	
Standard	10 - 11	25 - 28	MAS-S
Large	12 - 13	30 - 33	MAS-L

**LABELLING SYMBOLOGY**  
 EN ISO 15223-1:2021, unless otherwise stated

	Manufacturer		Consult Instructions for Use		Unique Device Identifier
	Authorised representative in the European Community		Caution		Single Patient Use
	Medical Device		Non-Sterile		Quantity - Abbreviation
	Catalogue Number		CE Mark - EU Regulation 2017/745		Contains Latex
	Batch Code		UKCA Mark - UK Regulation SI 2002 No 618		Date of Manufacture

MAS IFU - Revision 5 - September 24

**CHANECO LTD**  
 4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
 T: 01604 709999 F: 01604 700057



Ukalia Ltd  
 Clockwise River, 48-60 King St,  
 Belfast, BT1 2BE  
 Northern Ireland  
 info@ukalia.com

