



The following should be read thoroughly read before use:

### Important Notes:

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

### Intended User:

For use by both suitably trained healthcare professionals in a healthcare establishment and 'Lay Users' in a home use environment.

### Intended Purpose:

Intended to support lumbar spine by providing compression and stabilisation.

### Indications For Use:

- Low back pain
- Lumbar sprains and strains
- Post-trauma/surgery
- Patients who find conventional corsets unpleasant to wear

### Product Characteristics / Features:

- Constructed of vented, breathable, elastic stretch material
- Depth 10" (25cm) back and 7½" (18cm) front
- Durable elastic double fulcrum pulls for additional compression and support
- Four rigid stays provide firm circumferential support
- Secure touch and close fastening

### How it's supplied:

It is supplied non-sterile and ready for single patient use.

### Devices Required For Use, And Interoperability

#### Requirements:

None

### Storage Conditions:

Store in a cool and dry environment.

### Care and Maintenance:

- Wash in cold water, gentle cycle and mild soap
- Remove metal stays before washing
- Allow to dry naturally
- Do not expose to alcohol, ointments or solvents
- The device should be safety checked by the providing service and/or user every 6 months.

### Contraindications:

None

### Precautions / Warnings / Residual Risks:

Contains latex.

If allergic skin rashes occur, please consult your healthcare professional

### Possible Adverse Reactions:

Contains latex.

### Single Patient Use Precaution:

This device is designed and sold for single patient use only. The effects of any unauthorised reuse can result in the following complications:

- Risk of Cross contamination;
- Risk of mechanical fatigue, and/or associated failure.

### Directions For Use / Fitting Instructions:

- Open out the abdominal support
- Position the support around your waist
- Wrap the support around your waist and secure using the hook and loop fastening

### Disposal

Discard after single patient use and dispose of as common waste according to national guidelines.
















### Serious Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### Sizes & REF / Codes

Size	Waist Circumference		Code
	Inches	CM	
Small	22 - 28	56 - 71	VLSS-S
Medium	27 - 33	69 - 84	VLSS-M
Large	32 - 38	81 - 96	VLSS-L
X Large	37 - 43	94 - 109	VLSS-XL
XX Large	42 - 48	107 - 122	VLSS-XXL

LABELLING SYMBOLOGY  
EN ISO 15223-1:2021, unless otherwise stated

	Manufacturer		Consult Instructions for Use		Unique Device Identifier
	Authorised representative in the European Community		Caution		Single Patient Use
	Medical Device		Non-Sterile		Quantity - Abbreviation
	Catalogue Number		CE Mark - EU Regulation 2017/745		Contains Latex
	Batch Code		UKCA Mark - UK Regulation SI 2002 No 618		Date of Manufacture

VLSS IFU - Revision 3 - September 2024

 **CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057

**UK  
CA**

**CE**

**EC REP**

Ukalia Ltd  
Clockwise River, 48-60 King St,  
Belfast, BT1 2BE  
Northern Ireland  
info@ukalia.com

