

The following should be read thoroughly read before use:

Important Notes:

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

Intended User:

For use by both suitably trained healthcare professionals in a healthcare establishment and 'Lay Users' in a home use environment.

Intended Purpose:

Provides comfort, compression, support and immobilization. Restricts movement of the first MCP and CMC joints.

Indications For Use:

- Post-surgery / cast removal
- Post-injury e.g. sprains and strains
- Carpometacarpal thumb instabilities
- R/A and O/A and other joint pain

Product Characteristics / Features:

- Neoprene wrist brace support with mouldable thumb extension stay
- Restricts movement of first MCP and CMC joints
- Secure wrapping strap at thumb section
- Hook and loop style closure for secure fit and ease of adjustment
- Overall length 6½" (16.5cm)

How it's supplied:

It is supplied non-sterile, ready for single patient multiple use.

Devices Required For Use, And Interoperability

Requirements:

None

Storage Conditions:

Store in a cool and dry environment.

Contraindications:

None

Care and Maintenance:

- Remove metal splint prior to washing
- Wash in cold water, gentle cycle and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature.
- Do not expose to alcohol, ointments or solvents.
- The device should be safety checked by the providing service and/or user every 6 months.

Precautions / Warnings / Residual Risks:

If allergic skin rashes occur, please consult your healthcare professional

Possible Adverse Reactions:

None

Single Patient Use Precaution:

This device is designed and sold for single patient use only.

The effects of any unauthorised reuse can result in the following complications:

Risk of Cross contamination;

Risk of mechanical fatigue, and/or associated failure.

Directions For Use / Fitting Instructions:

1. Measure the wrist just above the wrist bone and select the correct size.
2. Loosen all the straps and place the hand inside the brace, gently positioning the thumb in the support
3. Secure the straps firmly.

Serious Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
















Disposal

Discard and dispose of as common waste according to national guidelines.


Sizes & REF / Codes

Size	Wrist Circumference		Code	
	Inches	CM	Left	Right
X Small	4 - 5	10 - 12.5	WTBN-XS-L	WTBN-XS-R
Small	5 - 6	12.5 - 15	WTBN-S-L	WTBN-S-R
Medium	6 - 7	15 - 17.5	WTBN-M-L	WTBN-M-R
Large	7 - 8	17.5 - 20	WTBN-L-L	WTBN-L-R
X Large	8 - 9	20 - 22.5	WTBN-XL-L	WTBN-XL-R

LABELLING SYMBOLOGY
EN ISO 15223-1:2021, unless otherwise stated

	Manufacturer		Consult Instructions for Use		Unique Device Identifier
	Authorised representative in the European Community		Caution		Single Patient Use
	Medical Device		Non-Sterile		Quantity - Abbreviation
	Catalogue Number		CE Mark - EU Regulation 2017/745		Contains Latex
	Batch Code		UKCA Mark - UK Regulation SI 2002 No 618		Date of Manufacture

WTBN IFU - Revision 5 - September 24

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057



Ukalia Ltd
Clockwise River, 48-60 King St,
Belfast, BT1 2BE
Northern Ireland
info@ukalia.com

