



**Restorative Care of America**  
INCORPORATED

12221 33rd Street N, St Petersburg, FL 33615  
www.rcai.com


Literature may not reflect current  
manufacturer specifications.

2/07



## Abduction Bar 26AB

The Abduction Bar by RCAI is indicated postsurgically, or for conditions requiring abduction control of the hip. The adjustable bar of the brace provides abduction and adduction control during restricted ambulation. The nylon ball and socket joints of the Abduction Bar duplicate the anatomical motion of the hip joint. Breathable Neoprene liners prevent the brace from migrating on the legs.

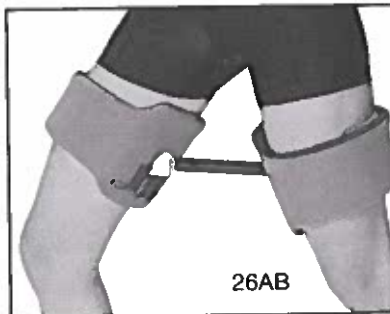

### Application

#### 1. Select proper size

Size	Measure mid thigh circumference	Narrow (adj. 5 1/4"–7") (13.3–17.8cm) Model # 26AB-A-N 26AB-P-N	Wide (adj. 7 3/4"–11 3/4") (19.7–29.8cm) Model # 26AB-A-W 26AB-P-W 26AB-P-XN
Adult	17 – 25" (43.2–63.5cm)		
Pediatric	12 – 18" (30.5–45.7cm)		
Ped. Ex	8 1/2 – 13" (21.6–33cm)	(3" not adj.)(7.6cm)	
Narrow	Adult Cuff height 6 1/2" (8.9cm)	Pediatric Cuff height 3 1/2" (8.9cm)	

#### 2. Passively stretch thigh muscles.

#### 3. Place brace between the legs and position at mid-thigh area.

#### 4. Secure Velcro cuff straps around patient's leg.

### Abduction Bar (without neoprene liner, use with Bilateral Hip Stabilizer)

Size	Measure mid thigh circ.	Narrow (adj. 5 1/4"–7") (13.3–17.8cm) Model # 26AB-A-N-OO 26AB-P-N-OO	Wide (adj. 7 3/4"–11 3/4") (19.7–29.8cm) Model # 26AB-A-W-OO 26AB-P-W-OO
Adult	17 – 25" (43.2–63.5cm)		
Pediatric	12 – 18" (30.5–45.7cm)		