



The following should be read thoroughly read before use:

Important Notes:

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

Intended User:

For use by both suitably trained healthcare professionals in a healthcare establishment and 'Lay Users' in a home use environment.

Intended Purpose:

Provides compression and heat retention.

Indications For Use:

- Mild strains and sprains
- Post-trauma
- Anterior knee pain
- Joint inflammation
- Chondromalacia patella

Product Characteristics / Features:

- Available in either grey neoprene with nylon lining or black Ortho-Tex
- Lateral and medial stays for stability and support
- Provides compression and heat retention
- Open patella with buttress pad to relieve pressure while providing direct contact and support
- Hook and loop straps above and below the knee to secure fit
- Universal - fits either left or right knee

How it's supplied:

It is supplied non-sterile, ready for single patient multiple use.

Devices Required For Use, And Interoperability

Requirements:

None

Storage Conditions:

Store in a cool and dry environment.

Contraindications:

None

Care and Maintenance:

- Wash in cold water, gentle cycle and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature.
- Do not expose to alcohol, ointments or solvents.
- The device should be safety checked by the providing service and/or user every 6 months.

Precautions / Warnings / Residual Risks:

If allergic skin rashes occur, please consult your healthcare professional

Possible Adverse Reactions:

None

Single Patient Use Precaution:

This device is designed and sold for single patient use only.

The effects of any unauthorised reuse can result in the following complications:

- Risk of Cross contamination;
- Risk of mechanical fatigue, and/or associated failure.

Directions For Use / Fitting Instructions:

1. Measure the knee circumference in full extension (straight leg) and refer to the size table to ensure the correct fit
2. Open both hook and loop straps
3. Gently pull on the support and slide up over the knee
4. Ensure that the patella ring is correctly positioned over the patella
5. Secure the hook and loops straps securely and comfortably

Serious Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
















Disposal

Discard and dispose of as common waste according to national guidelines.


Sizes & REF / Codes

Size	Knee Circumference		Code	
	Inches	CM	OrthoTex	Neoprene
Small	13 - 14	33 - 36	KSS-O-S	KSS-N-S
Medium	14 - 15	36 - 38	KSS-O-M	KSS-N-M
Large	15 - 16	38 - 41	KSS-O-L	KSS-N-L
X Large	16 - 18	41 - 46	KSS-O-XL	KSS-N-XL

LABELLING SYMBOLOGY
EN ISO 15223-1:2021, unless otherwise stated

	Manufacturer		Consult Instructions for Use		Unique Device Identifier
	Authorised representative in the European Community		Caution		Single Patient Use
	Medical Device		Non-Sterile		Quantity - Abbreviation
	Catalogue Number		CE Mark - EU Regulation 2017/745		Contains Latex
	Batch Code		UKCA Mark - UK Regulation SI 2002 No 618		Date of Manufacture

KSS IFU - Revision 5 - September 24

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057



Ukalia Ltd
Clockwise River, 48-60 King St,
Belfast, BT1 2BE
Northern Ireland
info@ukalia.com

