



The following should be read thoroughly before use:

Important Notes:

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

Intended User:

For use by both suitably trained healthcare professionals in a healthcare establishment and 'Lay Users' in a home use environment.

Intended Purpose:

Provides firm support to back and abdominal area during pregnancy.

Indications For Use:

- Pubic Symphysis pain
- Sacro-iliac pain
- Helps relieve lower back pain during pregnancy
- Can be used during post-natal recovery

Product Characteristics / Features:

- Elastic 3-strap design with secure hook and loop closure
- Allows the patient to obtain optimum fit through a range of strap positions, supporting the abdomen and reducing strain on the pelvis
- Strong lightweight elastic gives excellent support and comfort without being too restrictive
- Expands with increase in abdominal growth
- Easy to put on, take off and adjust whilst wearing

How it's supplied:

It is supplied non-sterile and ready for single patient use.

Devices Required For Use, And Interoperability Requirements:

None.

Storage Conditions:

Store in a cool and dry environment.

Care and Maintenance:

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not expose to alcohol, ointments or solvents
- The device should be safety checked by the providing service and/or user every 6 months.

Contraindications:

None.

Precautions / Warnings / Residual Risks:

Contains latex.

If allergic skin rashes occur, please consult your healthcare professional.

Possible Adverse Reactions:

Contains latex.

If allergic skin rashes occur, please consult your healthcare professional.

Single Patient Use Precaution:

This device is designed and sold for single patient use only. The effects of any unauthorised reuse can result in the following complications:

Risk of Cross contamination;

Risk of mechanical fatigue, and/or associated failure.

Directions For Use / Fitting Instructions:

- Place the maternity belt over the sacro-iliac joints at the back and wrap the straps around the pelvis, directing them forwards and downwards to lie under the lower abdomen and across the pubic bone.
- You can position the three straps all together on top of each other for greater support or place the three strap in different positions to cover a wider range of support if required.
- Close at the front using the hook and loop fastening
- Finally make sure the maternity belt has enough support for the patient and that it is not too tight

Disposal

Discard after single patient use and dispose of as common waste according to national guidelines.

Serious Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.



Sizes & REF / Codes

Size	Abdomen Circumference		Code
	Inches	CM	
Small	35½ - 40¼	90 - 102	MBTS-S
Medium	40¼ - 45¼	102 - 116	MBTS-M
Large	45¼ - 53½	116 - 136	MBTS-L
X Large	53½ - 61½	136 - 156	MBTS-XL

EXPLANATION OF LABELLING SYMBOLOGY EN ISO 15223-1:2021, unless otherwise stated

	Manufacturer		Consult Instructions for Use		Unique Device Identifier
	Authorised representative in the European Community		Caution		Single Patient Use
	Medical Device		Non-Sterile		Quantity - Abbreviation
	Catalogue Number		CE Mark - EU Regulation 2017/745		Contains Latex
	Batch Code		UKCA Mark - UK Regulation SI 2002 No 618		Date of Manufacture

MBTS IFU—Revision No: 5 - December 2025

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 E: info@chaneco.co.uk



Ukalia Ltd
Clockwise River, 48-60 King St,
Belfast, BT1 2BE
Northern Ireland
info@ukalia.com

