

Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally
- Do not use bleach



3 Panel Abdominal Binder Fitting Guidelines & Care Instructions



AB3 IFU - Revision 5 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Features

- Constructed of high quality elastic stretch material
- 9" (23cm) depth
- Provides compression and support to abdominal area
- Touch and close secure fastening
- Option of an added bespoke hernia pad available on request

Indications

- Post-surgery
- Abdominal hernia
- Weak abdominal muscles

Size Table

Size	Waist Circumference		Code
	Inches	CM	
Small	22 - 30	55 - 75	AB3-XS
Medium	26 - 34	65 - 85	AB3-M
Large	30 - 38	80 - 95	AB3-L
X Large	36 - 42	95 - 105	AB3-XL
XX-Large	38 - 46	105 - 120	AB3-XXL

Fitting Instructions

- Open out the abdominal support
- Position the support.....
- Wrap the support around your waist and secure using the hook and loop fastening

Additional Information

- Material content: Polyester 55%, Rubber 37%, Nylon 8%
- NOT LATEX FREE