

Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Foam Ankle Brace Fitting Guidelines & Care Instructions



ABF IFU - Revision 4 - Jan 2024

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

Lightweight and breathable, the Foam Ankle Brace is made from two thermoplastic outer shells with padded foam cushions with memory for enhanced comfort. The ankle is stabilised and protected during normal walking motion.

- Thermoplastic stirrup brace provides lateral and medial ankle immobilisation while allowing normal dorsiflexion and plantarflexion
- Low profile, lightweight and breathable, with a padded foam cushion with memory for enhanced comfort
- Simple to apply and adjust with 2 hook and loop straps, which may be moved to wrap at any desired height
- Universal - fits left or right leg

Indications

- Sprained ankle
- Fibular or deltoid ligament ankle injuries
- Tendonitis
- Post-operative support

Size Table

Size	Code
Regular	ABF-R

Fitting Instructions

1. Open up all the Velcro straps
2. Regulate the length of the strap that joins the two halves of the brace and pass it under the heel
3. Position the halves with the foam cushions on each side of the ankle
4. Fasten the straps securely

Additional Information

- Material Content: Plastic Shell - 100% PE, Liner - Velour & PU Foam
- LATEX FREE