

## Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



## Ankle Brace with Strap

### Fitting Guidelines & Care Instructions



ABS IFU - Revision 5 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



## Features

- 1/8" (3mm) thick neoprene brace with nylon lining to prevent over-inversion or plantarflexion
- Closed cell material increases blood circulation and provides a warming effect
- Elastic strap provides added support and compression and allows for easy adjustment of tension
- Universal - fits either left or right ankle

## Indications

- Ankle pain
- Minor sprain injuries

## Size Table

Size	Shoe Size		Code
	UK	EURO	
Small	6 - 8	15 - 20	ABS-S
Medium	8 - 10	20 - 25	ABS-M
Large	10 - 12	25 - 30	ABS-L
X Large	12 - 14	30 - 35	ABS-XL

## Fitting Instructions

- Refer to the size table to ensure the correct fit
- Gently pull on the support and slide up over the foot
- Ensure heel area is correctly placed in the cut out area
- Wrap the strap around the foot and carefully secure using the hook and loop fastening



## Additional Information

- Material content: Neoprene 76.5%, Nylon 23.5%
- NOT LATEX FREE