

Care and Maintenance

- Wash periodically by hand with a damp cloth and a mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Paediatric Ankle Splint Fitting Guidelines & Care Instructions



ASP IFU - Revision 4 - Jan 2024

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Constructed from soft felt material
- Stays can be easily contoured for customised fitting
- Hook and loop straps for secure fitting
- Universal - fits either left or right ankle

Indications

- Strains and Sprains
- Post injury/surgery
- Night resting splint
- Juvenile RA

Size Table

Size	Max Ankle Circ.		Code
	Inches	CM	
Small	6¼	16	ASP-S
Medium	8	20	ASP-M
Large	9½	24	ASP-L

Fitting Instructions

- Refer to the size table to ensure the correct fit
- Undo all the straps and place the foot into the support
- Ensure heel area is correctly placed in the cut out
- Fasten all straps securely

Additional Information

- Material content: Felt, Nylon webbing, Cotton binding, 25mm plastic loops, aluminium stays
- LATEX FREE