

## Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Remove metal stays before washing
- Do not dry clean or tumble dry - allow to dry naturally
- Do not use bleach



## Deluxe Lumbar Sacral Support Fitting Guidelines & Care Instructions



DLSS IFU - Revision 5 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



## Features

- Constructed of high quality elastic stretch material
- Available with back depth of either 10" or 12"
- Provides support and compression to stabilise the lumbar, sacral and abdominal areas
- Four anatomically contoured steel stays
- Double fulcrum pulls add increased support
- Hook and loop closure for easy adjustment
- Breathable, open-cell foam material

## Indications


- General low back pain
- Lumbar spine strains, sprains and sciatica
- Post trauma / surgery
- Prolapsed disc
- OA / RA spines

## Size Table

Size	Waist Circumference (CM)		Code	
	10" Depth	12" Depth	10" Depth	12" Depth
Small	56 - 71	71 - 81	DLSS-S	LSS-S
Medium	69 - 84	81 - 91	DLSS-M	LSS-M
Large	81 - 96	91 - 102	DLSS-L	LSS-L
X Large	94 - 109	102 - 112	DLSS-XL	LSS-XL
XX Large	107 - 122	112 - 122	DLSS-XXL	LSS-XXL

## Fitting Instructions

- Open out the abdominal support
- Position the support around your waist
- Wrap the support around your waist and secure using the hook and loop fastening

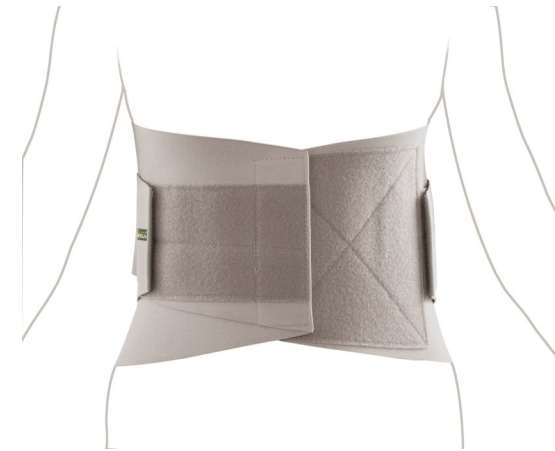

## Additional Information

- Material content: Polyester 35%, Latex 30%, Nylon 18%, Rayon 17%
- NOT LATEX FREE