

## Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



## Elevator Arm Sling Fitting Guidelines & Care Instructions



EAS IFU - Revision 5 - Jan 2024

 **CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire  
Retardant



## Features

- Adjustable sling for both elevation positioning and normal sling applications
- Manufactured from breathable foam laminate
- Easily converts to a sling for when the patient is ambulatory
- Shoulder pad for additional patient comfort
- Slip resist buckles ensure secure application
- Universal - fits right or left arm

## Indications


- Post-trauma
- Post-surgery
- Oedema control

## Size Table

Size	Forearm Length		Code
	Inches	CM	
Small	10 -11	25 - 28	EAS-S
Medium	11 - 12	28 - 30	EAS-M
Large	12 - 13	30 - 33	EAS-L
X Large	13 - 14	33 - 36	EAS-XL

## Fitting Instructions

- Detach the long piece of Velcro from the sling. This can be used to secure the shoulder strap of the sling to stop it from dangling down when the patient's arm is being elevated. This piece of Velcro can also be used to extend the elevated strap to reach any drip stand hook / correct elevated position required.
- Open out all the Velcro straps so the sling is ready for the arm to be placed inside.
- Place the arm into the sling by feeding the wrist through the attached wrist strap of the main sling. Make sure the elbow is seated snugly into the corner of the sling and close the straps around the arm.
- If using the sling to elevate the arm, your medical professional will elevate your sling via the horizontal strap, located over the hand.
- If using as a normal sling, the vertical shoulder strap will go over your shoulder and be clipped to the attached strap at the wrist area of the sling. The shoulder strap will then be adjusted to get the correct position required.

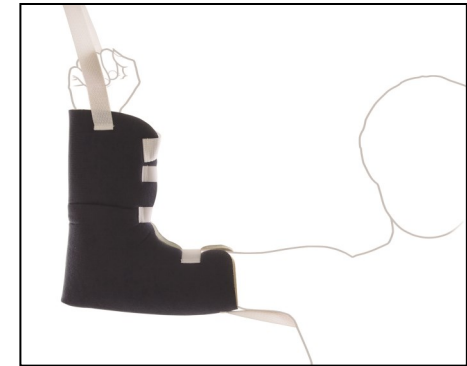

## Additional Information

- Material content: PU Foam 70%, PP 20%, Nylon 10%
- LATEX FREE