


## Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



## Four-Way Elastic Elbow Brace Fitting Guidelines & Care Instructions

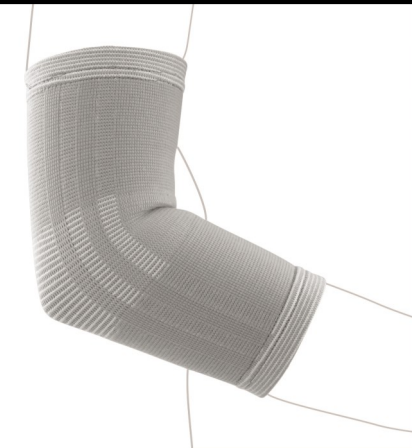

EEB IFU - Revision 4 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



## Features

- Provides firm support and compression to joints and muscles and helps to reduce swelling
- Four-way stretch material constructed from 65% nylon, 10% spandex and 25% rubber
- Anatomically designed for a better fit
- Universal - fits either left or right arm

## Indications

- Pain management
- Mild strains
- May help to prevent sporting injuries

## Size Table

Size	Forearm Circumference		Code
	Inches	CM	
Small	8 - 9	20 - 22	EEB-S
Medium	9 - 10	22 - 25	EEB-M
Large	10 - 11	25 - 28	EEB-L
X Large	11 - 12	28 - 30	EEB-XL

## Fitting Instructions

- Refer to the size table to ensure the correct fit
- Gently pull on the support and slide up over the elbow
- Make sure that the support does not roll down during use - re-adjust as necessary during the day

## Additional Information

- Material content: Nylon 65%, Rubber 25%, Spandex 10%
- NOT LATEX FREE