

Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Fixed Hip Abduction Splint

Fitting Guidelines & Care Instructions



FHAS IFU - Revision 5 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Polypropylene / Polyethylene construction, lined with closed cell foam and secure hook and loop fastening
- Abducts and flexes the hip joint, holding the head of the femur in a stable position
- Provides 90° hip flexion and 60° fixed hip abduction
- Easy to clean

Indications

- Congenital dislocation of the hip

Size Table

Size	Age (months)	Weight (lbs)	Waist Circ.		Mid Thigh Circ.		Code
			Inches	CM	Inches	CM	
X Small	<3	6 - 12	13 - 17	33 - 43	10	25	FHAS-XL
Small	3 - 6	13 - 16	14 - 18	36 - 46	11	28	FHAS-S
Medium	6 - 12	17 - 21	16 - 20	41 - 51	13	33	FHAS-M
Large	12 - 18	22 - 30	17 - 21	43 - 53	15	38	FHAS-L

Fitting Instructions

- Open the Velcro straps on the brace
- Place the patient into the brace and fold over the padded comfort protectors around each leg
- Secure the waist and leg straps
- If necessary, trim the padding with a pair of scissors to alleviate any area that you feel may cause rubbing

Additional Information

- Material content: PE 80%, Nylon 20%
- LATEX FREE