

Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



Four-Way Elastic Knee Support

Fitting Guidelines & Care Instructions



KSE IFU - Revision 5 - Jan 2024

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire
Retardant



Features

- Provides firm support and compression to joints and muscles and helps to reduce swelling
- Four-way stretch material constructed from 65% nylon, 10% spandex and 25% rubber
- Anatomically designed for a better fit
- Universal - fits either left or right knee

Indications

- Pain management
- Mild strains and sprains
- O/A and R/A mild knee support
- May help to prevent sporting injuries

Size Table

Size	Knee Circumference		Code
	Inches	CM	
Small	13 - 14	33 - 36	KSE-S
Medium	14 - 15	36 - 38	KSE-M
Large	15 - 16	38 - 41	KSE-L
X Large	16 - 18	41 - 46	KSE-XL

Fitting Instructions

- Refer to the size table to ensure the correct fit
- Gently pull on the support and slide up over the knee
- Make sure that the support does not roll down during use - re-adjust as necessary during the day

Additional Information

- Material content: Nylon 65%, Rubber 25%, Spandex 10%
- NOT LATEX FREE