

Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



Padded Knee Sleeve Fitting Guidelines & Care Instructions



KSP IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Available in either 1/8" (3mm) thick neoprene with nylon lining or black Ortho-Tex
- Overall length 12" (30.5cm)
- Provides compression and heat retention
- Open patella design for pressure relief with a neoprene pad for extra protection
- Universal - fits either left or right knee

Indications

- Mild strains and sprains
- Post-trauma
- Anterior knee pain
- O/A and R/A joint inflammation
- May help prevent injury

Size Table

Size	Knee Circumference		Code	
	Inches	CM	Ortho-Tex	Neoprene
Small	13 - 14	33 - 36	KSP-O-S	KSP-N-S
Medium	14 - 15	36 - 38	KSP-O-M	KSP-N-M
Large	15 - 16	38 - 41	KSP-O-L	KSP-S-L
X Large	16 - 18	41 - 46	KSP-O-XL	KSP-N-XL

Fitting Instructions

- Measure the knee circumference in full extension (straight leg) and refer to the size table to ensure the correct fit
- Gently pull on the support and slide up over the knee
- Ensure that the patella ring is correctly positioned over the patella
- Make sure that the support does not roll down during use - re-adjust as necessary during the day

Additional Information

- Material content: 76.5% Blended Neoprene, 23.5% Nylon
- LATEX FREE