

## Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



## Patella Stabiliser

### Fitting Guidelines & Care Instructions



KSS IFU - Revision 4 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



## Features

- Available in either grey neoprene with nylon lining or black Ortho-Tex
- Lateral and medial stays for stability and support
- Provides compression and heat retention
- Open patella with buttress pad to relieve pressure while providing direct contact and support
- Hook and loop straps above and below the knee to secure fit
- Universal - fits either left or right knee

## Indications

- Mild strains and sprains
- Post-trauma
- Anterior knee pain
- Joint inflammation
- Chondromalacia patella

## Size Table

Size	Knee Circumference		Code	
	Inches	CM	Ortho-Tex	Neoprene
Small	13 - 14	33 - 36	KSS-O-S	KSS-N-S
Medium	14 - 15	36 - 38	KSS-O-M	KSS-N-M
Large	15 - 16	38 - 41	KSS-O-L	KSS-S-L
X Large	16 - 18	41 - 46	KSS-O-XL	KSS-N-XL

## Fitting Instructions

- Measure the knee circumference in full extension (straight leg) and refer to the size table to ensure the correct fit
- Open both hook and loop straps
- Gently pull on the support and slide up over the knee
- Ensure that the patella ring is correctly positioned over the patella
- Secure the hook and loops straps securely and comfortably

## Additional Information

- Material content: 76.5% Blended Neoprene, 23.5% Nylon
- LATEX FREE