

Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



Universal Knee Wrap Fitting Guidelines & Care Instructions



KW IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Features

- Available in either grey neoprene with plush cloth lining or black Ortho-TEX
- Open patella design
- Provides knee support and protection
- Hook and loop closure for easy application and allows adjustment for better fit
- Universal - fits either left or right knee

Indications

- Mild strains and sprains
- Knee pain

Size Table

Size	Code	
	Ortho-TEX	Neoprene
Universal	KW-O-UNI	KW-N-UNI

Fitting Instructions

- Undo the hook and loop straps
- Wrap around the leg, making sure the knee cap is visible through the opening
- Fasten the wrap using the hook and loop straps to ensure a snug fit

Additional Information

- Material content: 76.5% Blended Neoprene, 23.5% Nylon
- LATEX FREE