

Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Deluxe Maternity Belt Fitting Guidelines & Care Instructions



MBD IFU - Revision 5 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Easily adjustable contoured support belt with hook and loop closure
- This maternity belt is made from durable elastic strapping material for the patient comfort and support
- The depth of the belt at the rear is 15cm which tapers to 13.5cm at the front to fit snugly under the patients tummy to help to support it
- Low profile shape for improved fit when sitting

Indications

- Pubic Symphysis pain
- Sacro-iliac pain
- Helps relieve lower back pain during pregnancy
- Can be used during post-natal recovery

Size Table

Size	Abdomen Circ.		Code
	Inches	CM	
X Small	35 - 38	90 - 96	MBD-XS
Small	38 - 40	90 - 102	MBD-S
Medium	40 - 42	102 - 108	MBD-M
Large	42 - 46	108 - 116	MBD-L
X Large	46 - 49	116 - 126	MBD-XL
XX Large	49 - 53	126 - 136	MBD-XXL
XXX Large	53 - 57	136 - 146	MBD-XXXL
XXXX Large	57 - 61	146 - 156	MBD-XXXXL

Fitting Instructions

- Place the maternity belt over the sacro-iliac joints at the back and wrap the maternity belt around the pelvis, directing either side of the maternity belt around the lower abdomen and across the pubic bone
- Close the belt at the front using the hook and loop fastening
- Finally make sure the maternity belt has enough support for the patient and that it is not too tight

Additional Information

- Material content: 80% PU Foam, 20% Nylon
- Contains latex