


Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Mallet Finger Splints Fitting Guidelines & Care Instructions

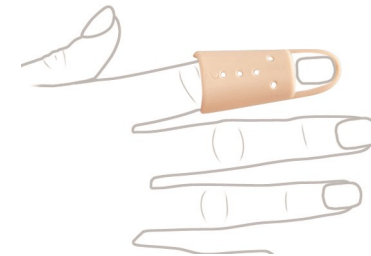

MFS IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Finger splint designed to support the DIP joint in neutral position
- Works to help correct mallet deformity
- Ventilated design for patient comfort
- Constructed of high quality, rigid plastic
- Sold in packs of 5

Indications

- Rehabilitation of mallet deformity

Size Table

Size	Joint Circ. (Middle of Finger)	Code
	CM	
1	4.5 - 5	MFS-1
2	5 - 5.5	MFS-2
3	5.5 - 6.5	MFS-3
4	6.5 - 7	MFS-4
5	7 - 7.5	MFS-5
6	7.5 - 8	MFS-6
7	8 - 8.5	MFS-7
8	8.5 - 9	MFS-8

Fitting Instructions

- Select the correct size from the Size Table
- Position the splint over the affected finger
- Secure the posterior end to the finger using adhesive tape

Additional Information

- Material content: 100% PE
- LATEX FREE