

## Care and Maintenance

- Wash periodically by hand with a damp cloth and a mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



## Multi Footwear Fitting Guidelines & Care Instructions



MUL IFU - Revision 4 - Jan 2024

 **CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire  
Retardant



## Features

- A range of fully adjustable temporary footwear, designed to provide maximum volume
- Available in Sandal, Shoe, Boot and Ultra Boot styles
- Black brushed nylon with polycotton lining
- Wide and Deep fitting
- Opens to toe (Ultra Boot also opens at the back for full accessibility)

## Indications

- To accommodate bandaged feet, oedema, deformities and prostheses
- Diabetic foot problems

## Size Table

UK Shoe Size	Multi Sandal	Multi Shoe	Multi Boot	Ultra Boot
1	MULSA-1	MULS-1	MULB-1	ULTB-1
2	MULSA-2	MULS-2	MULB-2	ULTB-2
3	MULSA-3	MULS-3	MULB-3	ULTB-3
4	MULSA-4	MULS-4	MULB-4	ULTB-4
5	MULSA-5	MULS-5	MULB-5	ULTB-5
6	MULSA-6	MULS-6	MULB-6	ULTB-6
7	MULSA-7	MULS-7	MULB-7	ULTB-7
8	MULSA-8	MULS-8	MULB-8	ULTB-8
9	MULSA-9	MULS-9	MULB-9	ULTB-9
10	MULSA-10	MULS-10	MULB-10	ULTB-10
11	MULSA-11	MULS-11	MULB-11	ULTB-11
12	MULSA-12	MULS-12	MULB-12	ULTB-12
13	MULSA-13	MULS-13	MULB-13	ULTB-13
14	MULSA-14	MULS-14	MULB-14	ULTB-14

## Fitting Instructions

- Open all the straps as fully as possible
- Carefully insert the foot into the footwear
- Close the straps securely and adjust for comfort

## Additional Information

- Material content: Outer - 100% Nylon, Lining - 100% Polyester, EVA Sole, 6mm Foam padding
- Removable Poron insole
- CONTAINS LATEX