

Care and Maintenance

- Wash periodically by hand with a damp cloth and a mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Ninewells Boot Fitting Guidelines & Care Instructions



NWB IFU - Revision 4 - Jan 2024

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire
Retardant



Features

- Unlined boot, opens to toe
- 3 combined hook and loop straps for complete adjustability and maximum volume
- Available in either black brushed nylon or neoprene
- Wide and Deep fitting with a wedge rocker sole
- Available in sizes 1 - 14

Indications

- To accommodate bandaged feet, oedema, deformities and prostheses
- Diabetic foot problems

Size Table

UK Shoe Size	Brushed Nylon	Neoprene
1	NWB-1	NWBN-1
2	NWB-2	NWBN-2
3	NWB-3	NWBN-3
4	NWB-4	NWBN-4
5	NWB-5	NWBN-5
6	NWB-6	NWBN-6
7	NWB-7	NWBN-7
8	NWB-8	NWBN-8
9	NWB-9	NWBN-9
10	NWB-10	NWBN-10
11	NWB-11	NWBN-11
12	NWB-12	NWBN-12
13	NWB-13	NWBN-13
14	NWB-14	NWBN-14

Fitting Instructions

- Open all the straps as fully as possible
- Carefully insert the foot into the footwear
- Close the straps securely and adjust for comfort

Additional Information

- Material content: Standard Outer - 100% Nylon, Neoprene Outer - 50% Polyamide, 50% Polyetha, Lining - 100% Polyamide
- Removable Poron insole
- CONTAINS LATEX