

Care and Maintenance

- Wash periodically by hand with a damp cloth and a mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Paediatric Post-Op Shoe with Ankle Strap Fitting Guidelines & Care Instructions



P-CB IFU - Revision 4 - Jan 2024

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Brushed nylon covered foam upper
- Moulded thermo plastic rubber sole
- Contoured back counter and ankle strap to prevent slippage
- Secure hook and loop straps
- Universal—fits left or right foot

Indications

- Post-surgery/trauma

Size Table

Size	Sole Length		Code
	Inches	CM	
Small	5½	14	P-CB-S
Medium	6¾	17	P-CB-M
Large	8	20	P-CB-L

Fitting Instructions

- Choose the correct size of post op shoe for your patient
- Open the two Velcro straps on the Post-Op Shoe
- Place the affected foot inside the shoe making sure the patient's heel is pushed back against the contoured heel counter
- Fasten the Velcro strap around the ankle first and then the top Velcro strap on the foot
- Finally check the two straps are secure and that they are not restricting blood flow around the foot

Additional Information

- Material Content: Upper - 100% Polyester, Polyurethane Foam, Insole - 100% Polyethylene, Outsole - 100% TPR
- LATEX FREE