

Care and Maintenance

- Wash periodically by hand with a damp cloth and a mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Paediatric Arm Sling Fitting Guidelines & Care Instructions



PAS IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire
Retardant



Features

- Brushed nylon covered foam
- Adjustable shoulder strap for patient comfort
- Machine washable
- Universal - fits left or right arm

Indications

- Post-surgery
- Post-injury
- Strains and sprains

Size Table

Size	Sling Length		Sling Depth		Code
	Inches	CM	Inches	CM	
Small	7½	19	4¼	10.5	PAS-S
Medium	9	23	4¾	12	PAS-M
Large	10½	27	5¼	13.5	PAS-L

Fitting Instructions

1. Place the injured arm into the sling, making sure the elbow and the back of the arm is sitting comfortably in the back of the sling.
2. Place the long strap around the neck so the padding sits around the side of the neck and the shoulder.
3. Guide the end of the strap through the bottom section of the metal clasp.
4. Bend the fabric strap so you can guide it back through the top section of the metal clasp.
5. Pull the strap upwards to secure.
6. Adjust the strap length until comfortable.

Additional Information

- Material Content: 4mm foam with brushed nylon, nylon webbing, 18mm cotton binding, plastic loops
- LATEX FREE