

## Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



## Paediatric Cast Boot

### Fitting Guidelines & Care Instructions



PCBO IFU - Revision 4 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



## Features

- Felt covered upper
- Moulded thermo plastic rubber sole
- Open toe design with open heel for ease of access
- Secure hook and loop straps
- Sizes Small to X Large —fits left or right foot

## Indications

- Post-surgery/trauma
- Early mobilisation

## Size Table

Size	Sole Length		Code
	CM	Inches	
Small	11.5	4½	PCBO-S
Medium	14	5½	PCBO-M
Large	16.5	6½	PCBO-L
X Large	18.5	7½	PCBO-XL

## Fitting Instructions

- Choose the correct size for your patient from the size table
- Open the three straps strap on the cast boot
- Place the affected foot inside the boot making sure the patient's heel is pushed back to the rear of the sole plate
- Fasten the strap at the back of the heel first and then the two top straps on the foot
- Finally check the three straps are secure and that they are not restricting blood flow around the foot

## Additional Information

- Material content: Upper - 100% Polyester, Straps - 100% Nylon, Insole - Cotton & Polyethylene, Outsole - TPR
- Latex Free