

## Care and Maintenance

- Wash periodically by hand with a damp cloth and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature.
- Do not hang up, or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight etc
- Do not expose to alcohol, ointments or solvents.



## Paediatric 3-Panel Knee Immobiliser Fitting Guidelines & Care Instructions



PKI-3P IFU - Revision 4 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire  
Retardant



## Features

- Brushed nylon covered foam
- Removable side panels can be relocated and angled for customised fitting
- Medial, lateral and three posterior stays
- Open patella area
- Secure hook and loop straps
- Universal - fits left or right knee
- Sizes Small to X Large

## Indications

- Juvenile R/A
- Night resting splint
- Strains & Sprains
- Post-surgery / trauma

## Size Table

Size	Overall Length		Code
	Inches	CM	
Small	7½	19	PKI-3P-S
Medium	8½	22	PKI-3P-M
Large	9½	24	PKI-3P-L
X Large	10½	27	PKI-3P-XL

## Fitting Instructions

- Open the two Velcro straps on the knee immobiliser and detach the two side panels
- Slide the main body of the knee immobiliser under the affected leg whilst the patient is laying flat and the leg is in extension
- Take each side panel and shape to the contour of the patient's medial and lateral side of their leg. Then attach the side panels to the main body of the knee immobiliser
- Fasten the top and bottom Velcro straps around the leg
- Finally check the two straps are secure and that they are not restricting blood flow around the leg

## Additional Information

- Material Content: 4mm foam with brushed nylon top and bottom, cotton binding and thread, aluminium stays, plastic loops
- LATEX FREE