



The following should be read thoroughly read before use:

Important Notes:

This document is designed to provide instructions on how this product/s is used. It is not a reference to clinical techniques, and does not replace local or national guidance or protocols.

Intended User:

For use by a suitably trained healthcare professionals in a healthcare establishment

Intended Purpose:

The Post-Op Shoe offers control and protection following surgery, without total immobilisation. Adjustable to accommodate bulky bandages.

Indications For Use:

- Any forefoot conditions e.g. Hammer Toes, Bunions, Neuroma etc
- Mid-foot procedures
- Osteotomies

Product Characteristics / Features:

Temporary Footwear

Key functional elements include:

- Padded collar to reduce slippage
- Hook and loop straps with D-ring for secure fastening
- Lightweight sole with padded breathable upper
- Removable tongue for optimum placement and comfort
- Sold singularly
- Latex free

How it's supplied:

It is supplied non-sterile and ready for single patient use.

Devices Required For Use, And Interoperability Requirements:

None

Storage Conditions:

Store in box, away from direct sunlight and heat sources in a cool and dry environment.

Care and Maintenance:

Wash periodically by hand with a damp cloth and a mild soap
 Use a towel to absorb most of the dampness and then allow to dry at room temperature
 Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
 Do not expose to alcohol, ointments or solvents
 The device should be safety checked by the providing service and/or user every 6 months.

Contraindications:

None

Precautions / Warnings / Residual Risks:

Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
 Do not expose to alcohol, ointments or solvents
 Do not use if package is damaged or broken.
 Keep dry and away from direct sunlight and heat sources in a cool and dry environment.
 If allergic skin rashes occur, please consult your healthcare professional

Possible Adverse Reactions:

None

Single Patient Use Precaution:

This device is designed and sold for single patient use only. The effects of any unauthorised reuse can result in the following complications:
 Risk of Cross contamination;
 Risk of mechanical fatigue, and/or associated failure.

Directions For Use / Fitting Instructions:

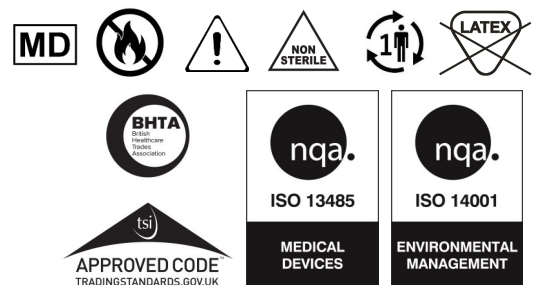
- Open all the straps as fully as possible
- Carefully insert the foot into the footwear making sure the heel is at the back of the shoe
- Close the straps securely but comfortably

Disposal

Discard after single patient use and dispose of as common waste according to national guidelines.

Serious Incident Reporting


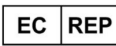



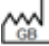












Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.



Sizes & Codes

Size	Sole Length (CM)	Shoe Size		Code
		UK	EU	
XX Small	24	1 - 3	33 - 36	POS-F-XXS
X Small	27	3 - 5	36 - 38	POS-F-XS
Small	28.5	5 - 6½	38 - 40	POS-F-S
Medium	29	6½ - 8	40 - 42	POS-F-M
Large	30	8 - 9½	42 - 44	POS-F-L
X Large	32	9½ - 11	44 - 46	POS-F-XL
XX Large	33.5	11 - 13½	46 - 48	POS-F-XXL

Labelling Symbolology - EN ISO 15223-1:2020, unless otherwise stated

			
Manufacturer	Authorized representative in the European Community	Catalogue number	Batch code
			
Contains Latex.	Manufactured in the United Kingdom of Great Britain and Northern Ireland on date	Caution	Consult Instructions For Use
			
Medical device	Unique Device Identifier	CE Mark - EU Regulation 2017/745	UKCA Mark - UK Regulation SI 2002 No 618
			
Keep away from sunlight	Keep dry	Non-sterile	Single patient use
QTY			
Quantity - abbreviation	Not Fire Retardant		Do not use if package is damaged

POS-F IFU - Revision 5 - January 2024

CHANECO LTD
 4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
 T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

