


Care and Maintenance

- Remove the metal stay from the brace before washing
- Wash in cold water, gentle cycle and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Paediatric Wrist Splint

Fitting Guidelines & Care Instructions

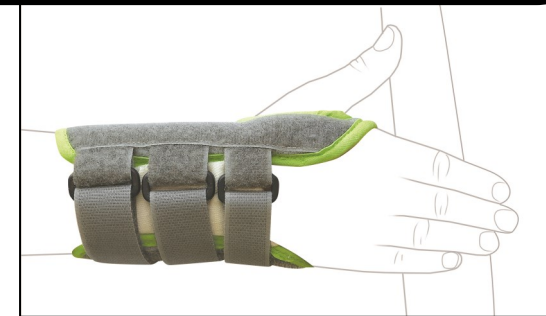

PWS IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Brushed nylon covered foam
- Three secure hook and loop straps
- Contoured design for better fit
- Removable palmar stay, easily shaped

Indications

- Post-surgery / cast removal
- Post-injury e.g. sprains and strains
- Juvenile R/A

Size Table

Size	Overall Length		Wrist Circumference		Code	
	Inches	CM	Inches	CM	Left	Right
Small	4¾	12	2¼ - 4	7 - 10	PWS-S-L	PWS-S-R
Large	6	15	4 - 5½	10 - 14	PWS-L-L	PWS-L-R

Fitting Instructions

- Measure the patient's wrist and select the correct size
- Undo the three Velcro straps and open up the wrist brace fully
- Glide the thumb through the small opening in the brace making sure the opening is facing upwards and not downwards
- Finally close the three Velcro straps and check that the brace is not too tight and adversely affecting the patient's circulation



Additional Information

- Material content: Foam with nylon webbing, cotton binding, plastic loops, aluminium stay
- LATEX FREE