


Care and Maintenance

- Remove metal splint prior to washing
- Wash in cold water, gentle cycle and mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Paediatric Wrist Thumb Splint

Fitting Guidelines & Care Instructions

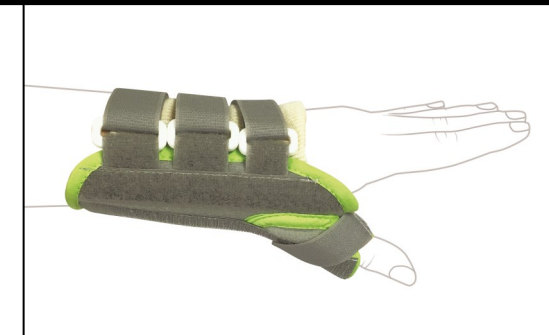

PWTS IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Brushed nylon covered foam
- Three secure hook and loop straps
- Contoured design for better fit
- Removable palmar stay, easily shaped

Indications

- Post-surgery / cast removal
- Post-injury e.g. sprains and strains
- Juvenile R/A

Size Table

| Size | Overall Length | | Wrist Circumference | | Code | |
|-------|----------------|----|---------------------|---------|----------|----------|
| | Inches | CM | Inches | CM | Left | Right |
| Small | 4¾ | 12 | 2¼ - 4 | 7 - 10 | PWTS-S-L | PWTS-S-R |
| Large | 6 | 15 | 4 - 5½ | 10 - 14 | PWTS-L-L | PWTS-L-R |

Fitting Instructions

- Measure the patient's wrist and select the correct size
- Undo all the Velcro straps and open up the brace fully
- Glide the thumb through the thumb support section making sure the opening is facing upwards and not downwards, and fasten the strap
- Finally close the three Velcro straps around the wrist section and check that the brace is not too tight and adversely affecting the patient's circulation



Additional Information

- Material content: Foam with nylon webbing, cotton binding, plastic loops, aluminium stay
- LATEX FREE