

Care and Maintenance

- Hand wash in warm water with mild soap
- Do not dry clean or tumble dry - allow to dry naturally, away from direct heat sources
- Do not use bleach



3/4 Length Silicone Insoles

Fitting Guidelines & Care Instructions



SI-34 IFU - Revision 4 - Jan 2024

CHANECO LTD
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



Features

- Dual density full length silicone insole with softer blue spot in heel and ball-of-foot area for additional cushioning
- Distributes even pressure over the full plantar surface
- Sold in pairs

Indications

- Pre/Post-surgery / trauma
- R/A and O/A
- Risk of skin or tissue breakdown
- General foot pain

Size Table

Size	Shoe Size		Code
	UK	Euro	
Small	<4	<37	SI-34-S
Medium	4 - 8	37 - 42	SI-34-M
Large	> 8	> 42	SI-34-L

Fitting Instructions

- Select the correct insoles from the sizes available – they can be trimmed using a sharp pair of scissors if required
- Open your shoes up as much as possible before inserting the insoles
- Place the insole heel first and push to the front of the shoe

Additional Information

- Material content: 100% silicone
- LATEX FREE