

## Care and Maintenance

- Hand wash in warm water and mild soap
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not use bleach



## Shoulder Immobiliser with Body Strap Fitting Guidelines & Care Instructions



SIB IFU - Revision 4 - Jan 2024

 **CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire  
Retardant



## Features

- Designed to immobilise the arm and shoulder
- Padded shoulder strap absorbs any pressure from the shoulder strap
- Thumb loop limits wrist drop and aids patient comfort
- Body strap helps control rotation
- Sizes Small to X-Large — fits left or right arm

## Indications

- Post-surgery/trauma
- CVA

## Size Table

Size	Forearm Length		Code
	Inches	CM	
Small	10 - 11	25 - 28	SIB-S
Medium	11 - 12	28 - 30	SIB-M
Large	12 - 13	30 - 33	SIB-L
X Large	13 - 14	33 - 36	SIB-XL

## Fitting Instructions

- Choose the correct size of sling for your patient from the size table
- Place the arm into the main body of the sling, making sure the elbow is seated right back against the seam of the sling
- Check the open end of the sling reaches the first knuckle to support the arm and the hand
- Fasten the shoulder strap and make sure the padded strap protector is positioned in the neck area for the patient's comfort
- Finally place the thumb into the thumb loop to prevent wrist drop and fasten the waist strap to prevent shoulder rotation



## Additional Information

- When choosing the correct size measure from the elbow to the first knuckle
- Material Content: 50% Polyester, 10% PU Foam, 15% Nylon, 5% ABS, 20% Cotton
- This product is latex free