

**Silicone Gel Toe Ring**  
**- Fitting Guidelines and Care Instructions**



**Features:**

- Pre-formed silicone gel toe ring
- Covers and protects the middle part of the toe
- Sold singularly

**Indications:**

- Corns and blisters
- Post-surgery scars
- Post-trauma

**Fitting Instructions:**

- Use as per clinician's guidelines

**Care & Maintenance:**

- Hand wash in warm water with mild soap
- Allow to dry naturally away from direct heat sources
- Do not use bleach

**Size Table**

Size	Code
Small/Medium	STR-S/M
Large/X-Large	STR-L/XL



CHANECO LTD  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
www.chaneco.co.uk  
Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



STR IFU - Revision 2- Jan 2024

