

Care and Maintenance

- Wash periodically by hand with a damp cloth and a mild soap
- Use a towel to absorb most of the dampness and then allow to dry at room temperature
- Do not dry clean or tumble dry - allow to dry naturally away from direct heat sources
- Do not expose to alcohol, ointments or solvents



Universal Night Splint Fitting Guidelines & Care Instructions



UNS IFU - Revision 4 - Jan 2024

 **CHANECO LTD**
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ
T: 01604 709999 F: 01604 700057
www.chaneco.co.uk

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire
Retardant



Features

- Maintains functional alignment of the foot and ankle
- Completely suspends and protects the heel from pressure or friction
- Tri-laminate material promotes air-circulation
- Hip rotation control bar
- Adjustable padded toe post for protection
- Padded ankle strap reinforces positioning

Indications

- Plantar Fasciitis
- Achilles Tendonitis
- Prevention of drop-foot in the immobile patient

Size Table

Size	Code
Universal	UNS

Fitting Instructions

- Open all the straps as fully as possible
- Carefully insert the foot into the orthosis, making sure your heel is at the back
- Close the straps securely but comfortably

Additional Information

- Material content: Outer Shell - PP 100%, Lining - Nylon 100%
- LATEX FREE